





ETERNA™ SPINAL CORD STIMULATION SYSTEM

QUICK START CHARGING GUIDE

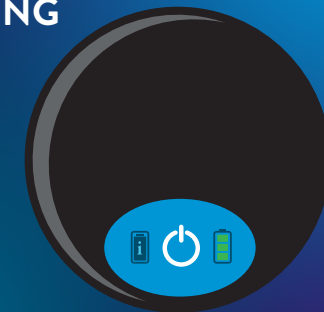


This quick start guide is not meant to be a substitute for reading the Charging System (Model 16000) User Guide for the Eterna™ SCS System.

Here is what you need to get started:

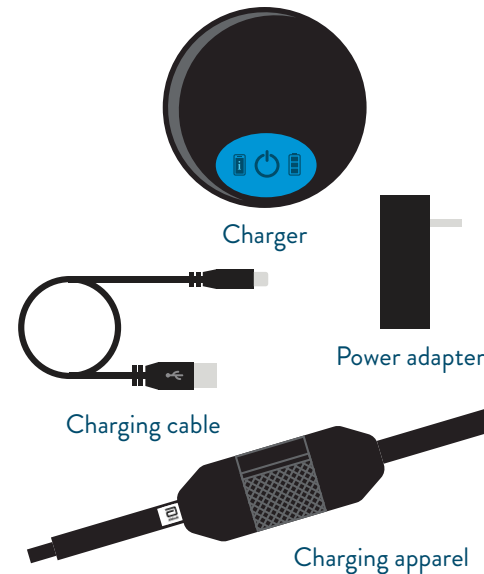


HOW TO CHARGE YOUR CHARGER

- 1 Plug the power adapter into an electrical outlet.

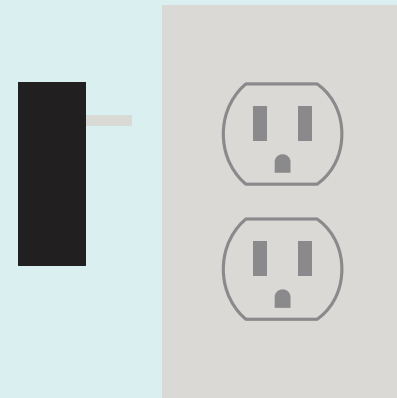


HOW TO CHARGE YOUR CHARGER

- 2 Place the charger on a stable surface like a table or on the floor.

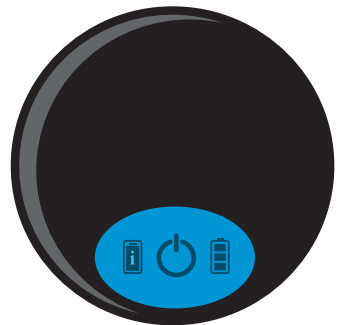


HOW TO CHARGE YOUR CHARGER

- 3 Connect the charging cable and power adapter.

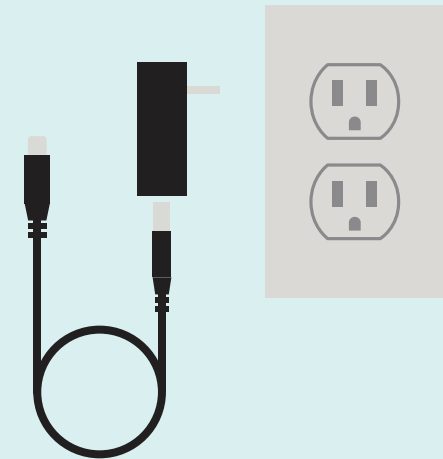


HOW TO CHARGE YOUR CHARGER

- 4 Plug charging cable into charger. Once it is fully charged, all segments of the battery icon will illuminate for 60 seconds and then turn off.

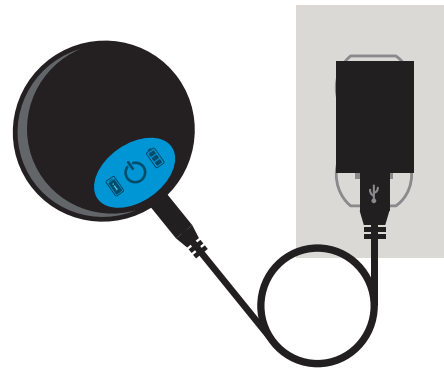

CHARGING YOUR GENERATOR

- 1 Unplug the charger from the charging cable. Press the power icon to turn on the charger. All three icons illuminate briefly and the charger beeps once if the sound is enabled.

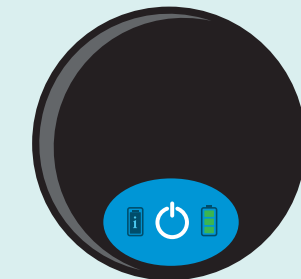

Note: While the charger is charging, it cannot be used to charge the generator.

CHARGING YOUR GENERATOR

- 2 Insert the charger fully into the mesh pocket of the charging apparel with the icons facing the opening of the pocket.




ONLY CHARGE GENERATOR WHILE YOU ARE AWAKE so that you can stop the charger if charging becomes uncomfortable.

Steps continue on reverse side →


CHARGING YOUR GENERATOR

- ③ Ensure the elastic **edges of the pocket overlap and fully cover the charger**, then wrap the charging apparel around your waist so the charger is over your generator.

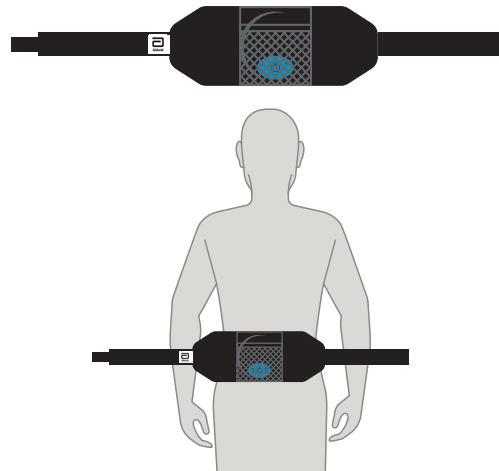


CHARGING YOUR GENERATOR

- ④ **Adjust the velcro strap so the apparel is secure** around your waist and the charger is on top of your generator.

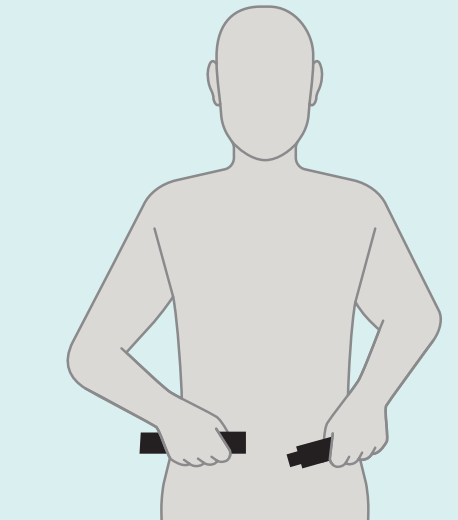

CHARGING YOUR GENERATOR

- ⑤ When the generator enters the charging zone, it **beeps continuously** until charging begins.

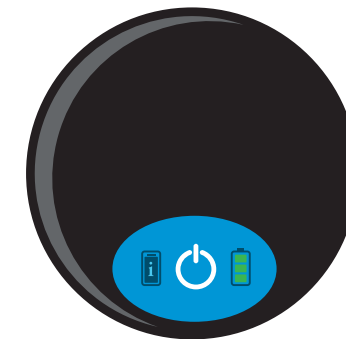

If the charger cannot find the generator within 90 seconds, it turns off automatically. If the charger turns off, press the power icon through the mesh pocket to turn the charger back on and realign it over your generator.


CHARGING YOUR GENERATOR

- ⑥ To confirm charging is occurring, note any one of the following:
- If the charger sound is enabled in the charger settings on the patient controller, **the charger beeps** when charging begins. When charging is in progress, the charger will not beep.
 - The charger power icon **begins blinking slowly**.
 - The patient controller displays the **charging status** and the generator **battery level**.

If you do not note any of these indicators, charging is **not occurring**.

CHARGING YOUR GENERATOR

- ⑦ When the generator is **fully charged, the charger beeps 3 times every 10 seconds** (if sound is enabled). Turn off the charger before removing the charging apparel. You can confirm that the generator is fully charged on your patient controller.

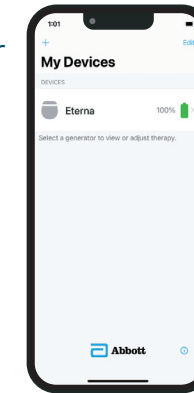

Charger Indicators

ICON	CHARGER LIGHT
	Power icon illuminates when the charger is turned on. When the charger is charging the generator, the power icon blinks slowly. When the charger is in pairing mode, the power icon blinks quickly.
	Battery icon illuminates in segments to indicate the charger battery level. While the charger is being charged, a segment of the battery icon blinks.

WARNING: You may perceive an increase in temperature at the generator site while you charge. Ensure that the area around the charger is well ventilated and not covered by an insulating material such as a blanket. If you feel discomfort while recharging your generator, stop using the charger and refer to “Troubleshooting” on page 16 of your Charging System Model 16000 User’s Guide. If the discomfort persists, contact your clinician. Only charge your generator while you are awake so you can stop the charger if charging becomes uncomfortable.

Charger Sounds

EVENT	CHARGER SOUND
Power on	1 beep
Volume on	1 beep
Power off	2 beeps
Error	3 beeps every 5 seconds
Low battery	3 beeps every 5 minutes
Generator found	short, repeating beeps
Generator charging started	2 beeps low tone then high tone
Generator charging stopped (not aligned)	2 beeps high tone then low tone
Generator charging complete (fully charged)	3 beeps low tone, high tone, then low tone

Abbott
8701 Bee Cave Road, Building 2 West, Austin, TX 78746 USA
Tel: 1 651 756 2000
Neuromodulation.Abbott

Rx Only

Brief Summary: Prior to using Abbott devices, please review the user manual for a complete list of indications, contraindications, warnings, precautions, potential adverse events, and directions for use. The charging system is intended to be used to charge the Eterna™ Implantable Pulse Generator.

™ Indicates a trademark of the Abbott group of companies.
© 2022 Abbott. All Rights Reserved.
MAT-2214861 v1.0 | Item approved for U.S. use only.

