



NEUROSPHERE™ VIRTUAL CLINIC

PATIENT
REGISTRATION
**QUICK START
GUIDE**

NEUROSPHERE™ DIGITAL CARE

Before your first NeuroSphere™ Virtual Clinic session, your doctor or Abbott representative will help you complete the registration process.

This guide will walk you through the required steps to enroll for NeuroSphere™ Virtual Clinic access and download Abbott's Patient Controller app to your Apple® mobile digital device (either an Abbott-provided iPod touch® mobile digital device or your personal iPhone® mobile digital device*).


WHAT YOU WILL NEED

To register for NeuroSphere™ Virtual Clinic access, you will need the following:

- iPod touch® or iPhone® mobile digital device that you use to access your Patient Controller app
 - Ensure this device is updated to a minimum iOS® version of 12.4 or higher. You can check the iOS version by clicking Settings > General > About > Software Version
- Internet connection
- Access to your email account
- Patient magnet for pairing

OPEN YOUR WELCOME EMAIL


You will receive a welcome email from donotreply@virtualclinic.abbott that outlines the three steps for completing your NeuroSphere™ Virtual Clinic registration.

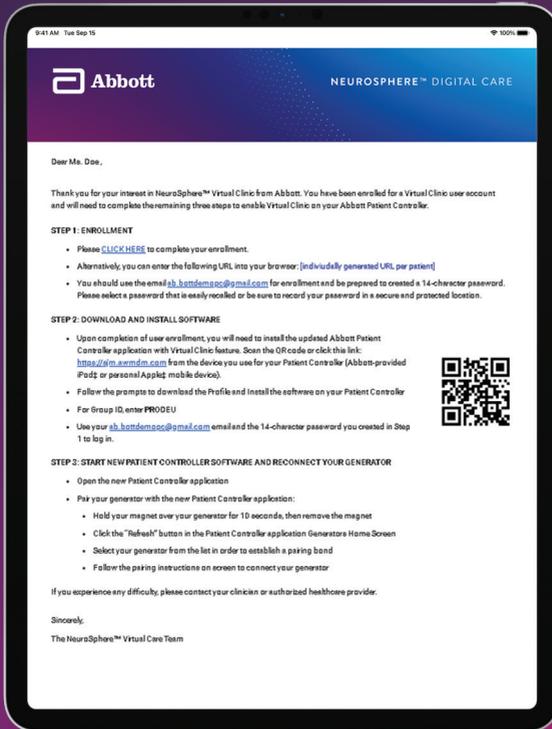

STEP ONE

A Navigate to the link provided under **“Step 1: Enrollment”** in the welcome email.


B Verify your account information by entering your primary phone number and your generator’s serial number.

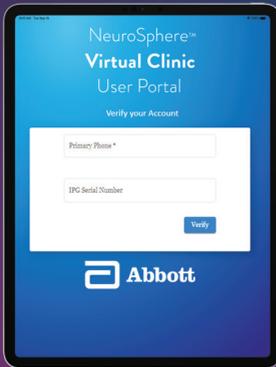

The serial number can be found on your Abbott-issued Patient Implant Card or by opening your existing Patient Controller app and tapping the **Information icon** on the therapy screen.

If your serial number includes a “.1” at the end, enter it without the “.1”; for example, **ABC123.1** should be entered as **ABC123**

C Review and accept the terms of use and privacy policy.

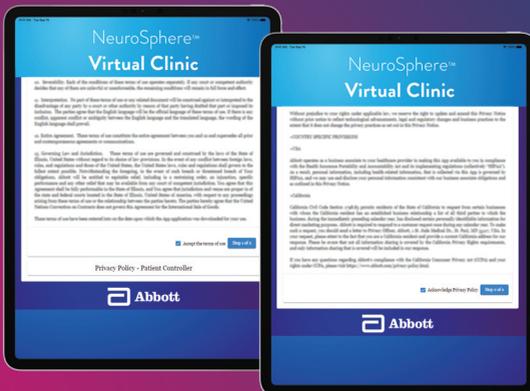

- D Create a 14-character password (we recommend using a phrase that is easy to remember), and tap **Save**.



- E Confirm your name is spelled correctly, and tap **Enroll**.



- F **You are now enrolled.** Close the browser window, and return to your welcome email to complete Step 2.



STEP TWO

Follow the instructions to download Abbott's Patient Controller app to your Apple[®] mobile digital device (either an Abbott-provided iPod touch[®] mobile digital device or your personal iPhone[®] mobile digital device).


A

Use your patient controller device's camera to scan this QR code, or navigate to this link: <https://sjm.awmdm.com>




B

For Group ID, enter **PRODEU** and tap **Next**.

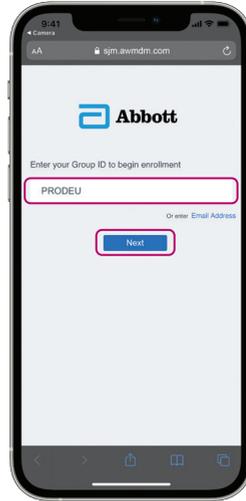


C

Enter your email and the 14-character password you created, and tap **Next**.

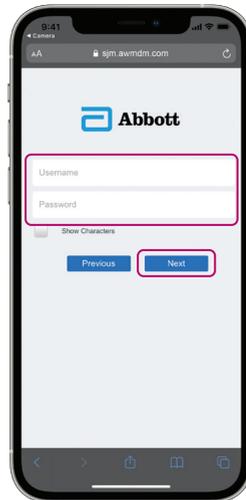


D Tap **Download Profile**.

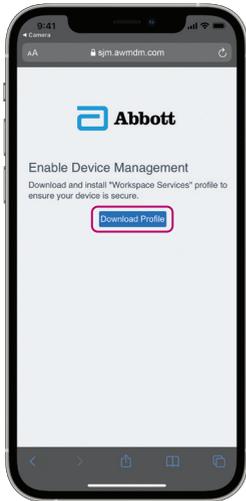


E Tap **Allow**.



F Tap **Close**.

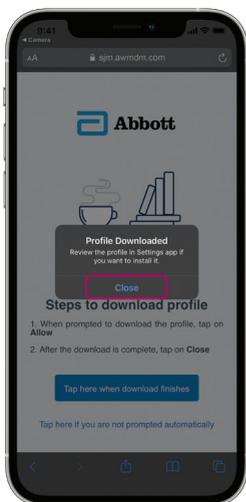

G When the download finishes, tap the button.

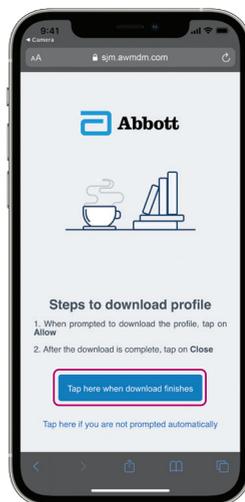


H On the home screen, tap the **Settings icon**.




I Tap **Profile Downloaded**.




J Tap **Install**.

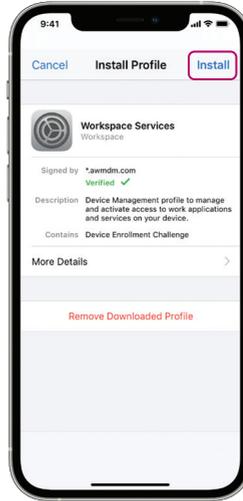


K Tap **Install**.

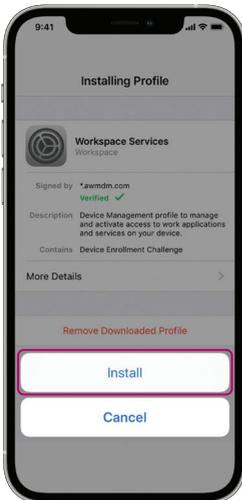


L Tap **Install**.

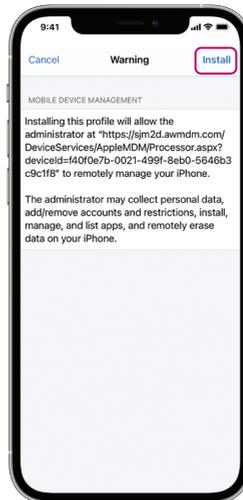

M Tap **Trust**.

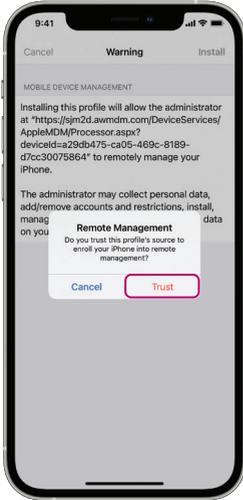

N Once the profile installation is complete, tap **Done**.

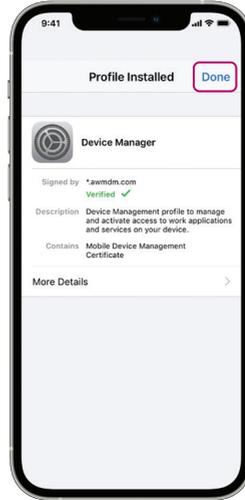

O Return to the home screen, and tap **Install** on the installation prompt (the prompt may take up to 30 seconds to appear).



The Patient Controller app will begin downloading.


The instructions on pages 10 and 11 of this guide should **only** be followed if you have a previous version of the Patient Controller app on the same mobile digital device you are enrolling for NeuroSphere™ Virtual Clinic access. If you do not have a previous version of the Patient Controller app, proceed to page 12.

DELETING YOUR PREVIOUS VERSION OF THE PATIENT CONTROLLER APP


- A** Locate the older Patient Controller app on the home screen of your mobile digital device.



- B** Hold down the app until this prompt appears, and then tap **Remove App**.

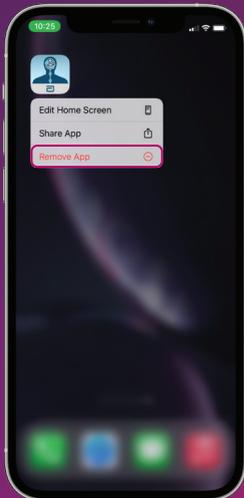

- C** Tap **Delete App**.

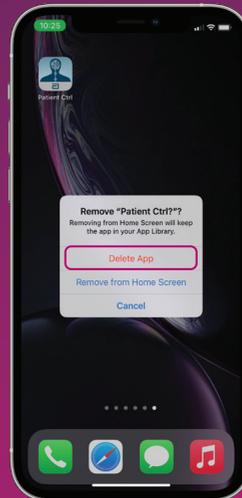

D

Go back to the home screen, and tap the **Settings icon**.




E

Tap **Bluetooth**.




F

Tap the **Information icon** to the right of “ORN.”

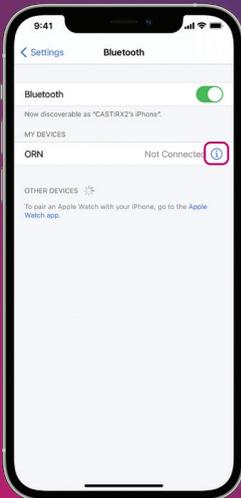

G

Tap **Forget This Device**.

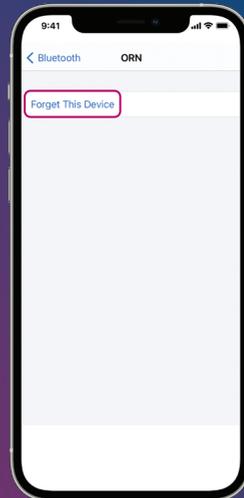


STEP THREE

Start the new Patient Controller app,
and reconnect your generator.


- A** Open your Patient Controller app.




- B** Hold the pairing magnet over your generator
for 10 seconds, then remove the pairing
magnet and tap the **Refresh icon**.

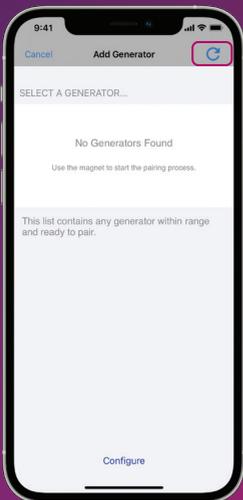

C Select your generator from the list.

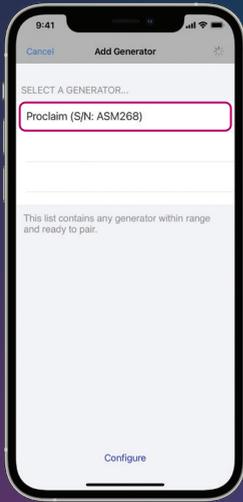

D Follow the on-screen pairing instructions to connect your generator.

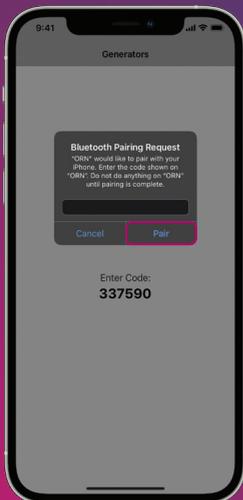

*Available on eligible Apple® mobile digital devices. For a list of personal Apple mobile devices compatible with Abbott's St. Jude Medical™ Patient Controller app, www.NMmobiledevicesync.com/int/dbs OR www.NMmobiledevicesync.com/int/ep.

Abbott

One St. Jude Medical Dr., St. Paul, MN 55117 USA, Tel: 1 651 756 2000
Neuromodulation.Abbott

Legal Manufacturer: Abbott Medical, 6901 Preston Road, Plano, Texas 75024, USA

Rx Only

Brief Summary: Prior to using Abbott devices, please review the Instructions for Use for a complete listing of indications, contraindications, warnings, precautions, potential adverse events and directions for use.

™ Indicates a trademark of the Abbott group of companies.

‡ Indicates a third party trademark, which is property of its respective owner. Bluetooth and Bluetooth logo are registered trademarks of Bluetooth SIG, Inc.
© 2024 Abbott. All Rights Reserved.
24-99839 MAT-2413678 v1.0 | Item approved for audiences in EMEA.

